
Quick Guide

spinit[®] BC



Quick Guide | How to perform a spinit® BC test?



Adhesive flap

Sample well

1



- Press the start button on spinit® screen and follow on screen instructions;
- After scanning the barcode from the disposable cartridge, prepare the disc:
- Take the disc out of the pouch and lift the adhesive flap;
- Place the disc on a plain surface with label facing up;

- Massage finger and disinfect the area before puncture;
- Prick finger and collect the 15 µL of blood with the appropriate capillary tube and allow blood to reach the indicator line;




NOT enough sample



Correct amount of sample

- Place blood into sample well and gently press the upper part of the capillary tube;
- After releasing the blood keep pressing the tube;
- After use, discard the tube appropriately;
- Seal the disc by removing flap protection and firmly close the adhesive flap

2

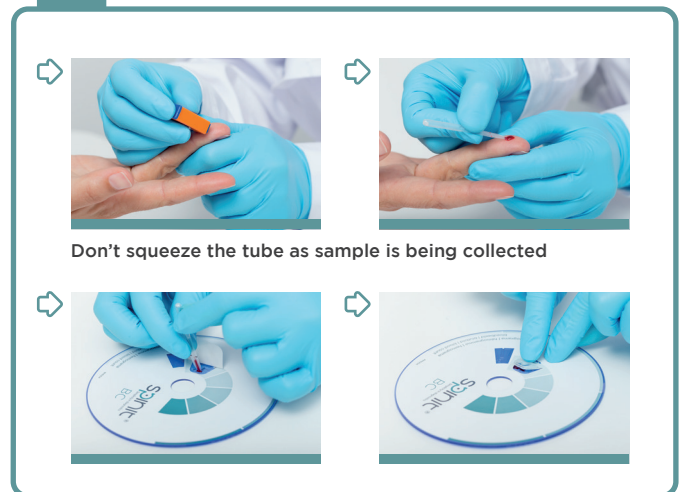

Don't squeeze the tube as sample is being collected

- Once you have the disc ready, click Next on the screen;
- Place the disc **immediately** into the instrument;
- Follow on-screen instructions;
- Wait for the test result;
- Discard disc appropriately

3

